Replacement Kit Instructions

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Safety Information

The following terms are used to describe potential hazards:

Warning:

Hazards or unsafe practices that may result in serious personal injury or death.

Caution:

Hazards or unsafe practices that may result in minor personal injury, product damage, or property damage.

WARNINGS

Possible LIFEPAK defibrillator shutdown.

- Always replace CHARGE-PAK battery charger and QUIK-PAK electrodes at the same time.

Possible Skin Burns or Ineffective Energy Delivery can occur due to air pockets, damaged, or dried out QUIK-PAK electrodes.

- Do not reuse electrode.
- Do not crush, fold, trim, or store electrode under heavy objects.

CAUTION

Risk of Fire and Burns or Explosion.

- Do not charge, do not open, crush, heat above 100°C (212°F) or incinerate the CHARGE-PAK battery charger. Follow manufacturer’s instructions.

Important!

- Do not open package until immediately before use.
- Before use, verify that electrode pouch seal is not damaged.
- Refer to your LIFEPAK CR Plus or LIFEPAK Express device operating instructions for complete directions for use, indications for use, contraindications, warnings, precautions, potential adverse effects, skin preparation, cleaning instructions, and device compatibility.
- The user and/or the patient should report any serious product-related incident to both the manufacturer and the local regulatory authority, such as the competent authority of the European Member State, where the user and/or patient is established.
- Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: CHARGE-PAK, LIFEPAK, LIFEPAK CR Plus, LIFEPAK Express, QUIK-PAK, and Stryker. All other trademarks are trademarks of their respective owners or holders. The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker’s trademark or other intellectual property rights concerning that name or logo.

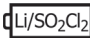
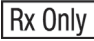
Use the Physio-Control replacement kit to replace the CHARGE-PAK battery charger and the QUIK-PAK electrode packet as follows:

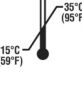
- After using the defibrillator
- If the CHARGE-PAK symbol appears in the readiness display
- When the Use By date is reached or passed

The replacement kit includes a CHARGE-PAK battery charger, one or two QUIK-PAK electrode packets, and a CHARGE-PAK battery discharger. The discharger depletes a used battery charger so that it is ready for recycling or disposal.

Note: Always keep the CHARGE-PAK battery charger with its original defibrillator. When a CHARGE-PAK battery charger is inserted into a defibrillator, it becomes internally linked to that device. Do not move the CHARGE-PAK charger to another defibrillator. If the CHARGE-PAK charger is removed from its original defibrillator and inserted into a second device, it will not charge and the CHARGE-PAK indicator will illuminate on the readiness display. If this occurs, discard the existing CHARGE-PAK battery charger and QUIK-PAK electrodes. Obtain a new replacement kit and replace both CHARGE-PAK battery charger and QUIK-PAK electrodes at the same time.

Follow the replacement instructions in the *LIFEPAK CR Plus Defibrillator/LIFEPAK EXPRESS Defibrillator Operating Instructions, Section 5.*

Symbol	Description	Symbol	Description	Symbol	Description
	Consult operating instructions		Caution		Type BF applied part
	CHARGE-PAK battery charger		Not made with natural rubber latex		Use By date shown: yyyy-mm-dd
	Direct current		Do not reuse		Manufacturer
	Date of manufacture		FIRE OR BURN HAZARD - Do not charge		FIRE OR BURN HAZARD - Do not deform or damage
	EXPLOSION HAZARD - Do not dispose of in fire		FIRE OR BURN HAZARD - Do not open or dismantle		Do not heat above 100°C (212°F)
	Recycle this item		Keep dry		Australian sponsor address
	Medical device		By prescription only		For USA audiences only

Symbol	Description	Symbol	Description	Symbol	Description
QTY	Quantity	PN	Part number	UPN	Universal product number
UDI	Unique device identifier	REF	Catalog number	PAT	Visit strykeremergencycare.com/patents for patent information
LOT	LOT number (for QUIK-PAK electrode) LOT number shown: YYWW (for CHARGE-PAK battery charger)		Do not dispose of this product in the unsorted municipal waste stream. Dispose of this product according to local regulations. See strykeremergencycare.com/recycling for instructions on the proper disposal of this product.		Storage at extreme temperatures of -40° or 70°C (-40° or 158°F) is limited to seven days. If storage at these temperatures exceeds one week, the electrode shelf-life will be reduced.
	Recommended storage temperature: 15° to 35°C (59° to 95°F).		Keep away from sunlight	CE	Mark of conformity to applicable European Regulations and / or Directives
EC REP	Authorized representative in the European Community	Assembled in the USA	Assembled in the USA	Assembled in Mexico	Assembled in Mexico

Specifications - QUIK-PAK Electrode

Characteristic	Description
Pads shelf life	Two years typical
Long-Term Storage Temperature	Electrodes to be stored in a cool, dry location (15° to 35°C) (59°F to 95°F).
Short-Term Storage Temperature	Shipping Temperature -40° to 70°C (-40° to 158°F), maximum exposure 7 days.

Specifications - CHARGE-PAK Battery Charger

Characteristic	Description
Type	Lithium Sulfuryl Chloride (Li/SO ₂ Cl ₂)
Voltage	11.7 V
Capacity	1.4 Ah
Standby Life	Replace after each use or when CHARGE-PAK indicator is visible (typically after 2 years)
Weight	80.5 grams (0.18 lb)
Operating Temperature	0° to 50°C (32° to 122°F)
Long-Term Storage Temperature	15° to 35°C (59° to 95°F)
Short-Term Storage Temperature	-40° to 70°C (-40° to 158°F) for a maximum of 1 week
Relative Humidity	5 to 95% non-condensing

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Physio-Control, Inc.,